Vetramil[®] PAD

THE NATURAL POWER OF HONEY

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Product

Vetramil[®] Pad
10x10 cm vochtig gaas

Samenstelling

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Bewaarcondities

Bewaren tussen 4 °C - 30 °C.

Patient Information Leaflet Vetramil Pad

Product

Vetramil[®] Pad
10x10 cm moist gauze

Ingredients

The fluid consists of: Ingredients: Aqua, Mel, Glycerin, Sorbitol, Potassium Sorbate.

Indication

Vetramil[®] Pad protects and soothes the irritated skin and has a positive effect on the natural healing process. Vetramil[®] Pad has a long-lasting effect and is suitable for all kinds of pets, both large and small.

Product description

Vetramil[®] Pad consists of a gauze that has been impregnated with a liquid based on natural honey. The gauze consists of an absorptive layer of natural viscose which has been coated with a layer of non-adherent polyethylene.

Beneficial effect

Vetramil[®] Pad is slightly moist and supports the natural healing process. The basic ingredients of the liquid soften the skin and protect against dehydration. The moist gauze covers the wound and can also absorb a small amount of extra moisture. Vetramil[®] Pad contains honey with various natural bee enzymes. These enzymes have a cleansing and purifying effect on the skin. The low pH value helps preserve the skin's natural acidity level.

How to apply

1. Clean the skin first (if necessary).
2. Open the packaging by tearing the notch in the sealed edge.
3. Remove the gauze from the packaging, open it out and apply it over the affected area of skin.
4. Affix the gauze to the skin using self-adhesive porous tape or a roller bandage if necessary.
5. Remove the dressing carefully after one to two days (maximum), and replace it with a new Vetramil[®] Pad dressing if necessary.

Contraindications

Do not use if the animal is hypersensitive to any of the product's ingredients.

Precautions

- Do not use if the packaging is damaged.
- Do not use if the packaging is not properly sealed.
- A gauze dressing should be used once only.

Storage conditions

Storage temperature: between 4 °C and 30 °C.

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Bfactory Health Products B.V.
FI: FaunaPharma Oy, Espoo - www.faunapharma.fi
DK: Axel Madsen A/S, Vedbæk - www.axel-madsen.dk
AT: Vana GmbH, 1020 Wien - www.vana.at
CH: Dr. E. Graeub AG, Bern - www.graeub.com